

FICHA TÉCNICA	
Product name	CRUTCH / MULETA FY925L(S)
Product description	Aluminum frame with matt silver color, grey plastic and yellow TPR handle and grip ,grey rubber tip
Composition formula	aluminum tube, rubber, plastic part, TPR
Product specifications	aluminum tube, rubber tip, 9 levers adjustment for down part tube, 5 levers adjustment for up part tube, Length: 20.5cm, Width: 5cm, height: 94-114cm, 20pcs/carton, N.W.: 14.6kg, G.W.: 16.7kg, package: 101*33*29cm
Uses and indications of	assist to walk
Instructions for use;	Adjust the stick to suitable height, adjusting method: press the spring lock with finger to adjust to the suitable height, fix the middle screw nut tightly, then it can be use. Examine whether the handle is loose or it has sting, prevent loose handle from endangering you, and be careful of the sting hurting your palms. Examine whether the anti-slip rubber tips of the stick are tight and stable. If they are worn, change them immediately. Pay more attention when use on the slope. Stick is the tool to aid walking, do not use it to support other things. Stick is not suitable for climbing mountains and supporting your body for a long time.

