

# HUAIAN WANJIA MEDICAL DEVICE CO., LTD.

淮 安 市 万 佳 医 用 器 械 有 限 公 司

ADDRESS: NO.1, GUOLINCHANG, SHUIDUKOU, QINGHE, HUAI'AN CITY, JIANGSU 223001, CHINA

Tel: 0086-517-84519182, 13515233016 Fax:0086-517-84519183

## Technical Sheet

### Product:

Disposable Blood Lancets

### Specification:

21G, 23G, 26G, 28G, 30G, 32G, 33G

### Packing:

100 pcs/ Box, 20,000pcs/ Carton

### Use:

Used to make punctures to obtain small blood specimens and to prick the skin testing for allergies.

### Material:

Body: PE, Needle: SUS304

### Method Of Sterility:

Sterilized by the Co-60 gamma radiation of 25kGy

### Manufacturer

HUAIAN WANJIA MEDICAL DEVICE CO., LTD.

ADDRESS: NO.1, GUOLINCHANG, SHUIDUKOU, QINGHE, HUAI'AN CITY, JIANGSU 223001, CHINA

### Standards


Is hereinafter confirmed to comply with the requirements set out in the Council Directive on the harmonization of the Laws of the Member States concerning Medical Device Directive (93/42/EEC, amended by 2007/47/EC)

The medical device has been assigned to class IIa according to Annex V of the Directive 93/42/EEC (amended by 2007/47/EC).

EN ISO14971: 2016, EN ISO11137-1:2015, EN ISO11607-1:2016, BS EN ISO7153-1-2016

EN ISO 15223-1:2016, EN1041: 2016

### Basic Dimensions:

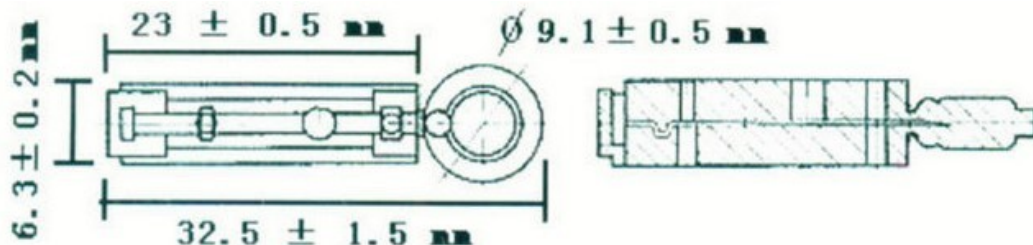

# **HUAIAN WANJIA MEDICAL DEVICE CO., LTD.**

淮 安 市 万 佳 医 用 器 械 有 限 公 司

---

ADDRESS: NO.1, GUOLINCHANG, SHUIDUKOU, QINGHE, HUAI'AN CITY, JIANGSU 223001, CHINA

Tel: 0086-517-84519182, 13515233016 Fax:0086-517-84519183

## **Instruction for use**

### **Operation method:**

1. Open the lancet protection cap;
2. Place the lancet in Lancing device
3. Launch the lancing device
4. Remove the lancets from lancing device and dispose

### **Announcements:**

1. Use the lancet as soon as it is unpacked.
2. The lancet is single-use, after usage it must be destroyed by burning or according to the ground of hospital to handle.

### **Storage:**

- 1.Avoid exposure to high temperature, high humidity, direct sunlight and water
- 2.Avoid scars and pinholes on the packing material.
- 3.Check the expiration date on the product. Discard if it is overdue. Also discard Lancets left unused longafter they are unpacked

### **Expiration Date:**

This product may be used up to 5 years after date of production, as indicated on the package label, so long as it is maintained in accordance with an appropriate storage procedure.

### **Warning:**

1. If package is broken, it is forbidden the usage.
2. If the point of the blood lancet is bent or broken, it is forbidden the usage.
- 3.The safety lancet must be used in the term of validity.
4. It must be disposed of as hospital waste, according to the provisions of the law